



TRUABUTMENT ASC TORQUE SPEC GUIDE

Ensure you are applying the right amount of torque for your patient's implant restoration.

Manufacturer	Brand	ASC* Recommended Screw Torque Value
Dentist	OneQ-SL/S-Clean Mini	15N. Cm
	OneQ-SL/S-Clean Regular	25N. Cm
	i-clean Regular	25N. Cm
Dentium	SuperLine (Implantium) Regular	25N. Cm
	SimpleLine II Regular	25N. Cm
	SimpleLine II Wide	25N. Cm
Dentsply Sirona	Astra Tech™ OsseoSpeed® EV 3.0	25N. Cm
	Astra Tech™ OsseoSpeed® EV 3.6	25N. Cm
	Astra Tech™ OsseoSpeed® EV 4.2	25N. Cm
	Astra Tech™ OsseoSpeed® EV 4.8	25N. Cm
	Astra Tech™ OsseoSpeed® EV 5.4	25N. Cm
	Astra Tech™ OsseoSpeed® TX 3.0	15N. Cm
	Astra Tech™ OsseoSpeed® TX 3.5, 4.0	20N. Cm
Astra Tech™ OsseoSpeed® TX 4.5, 5.0	25N. Cm	
DIO	UF Mini	15N. Cm
	UF Regular	25N. Cm
	UF Internal Regular	25N. Cm
	UF Internal Wide	25N. Cm
	SM Small	25N. Cm
	SM Regular	25N. Cm
Hiossen (Osstem)	Hiossen ET (Osstem ET) Mini	20N. Cm
	Hiossen ET (Osstem ET) Regular	25N. Cm
	Hiossen HG Ultra-Wide	25N. Cm
	Hiossen HS (Osstem SS) Regular Neck	25N. Cm
	Hiossen HS (Osstem SS) Wide	25N. Cm
Implant Direct	Legacy 3.0	25N. Cm
	Legacy 3.5	25N. Cm
	Legacy 4.5	25N. Cm
	Legacy 5.7	25N. Cm
	InterActive NP	25N. Cm
	InterActive RP	25N. Cm
Keystone Dental	PrimaConnex™/Genesis™ 3.8 SD	25N. Cm
	PrimaConnex™/Genesis™ 4.5 RD	25N. Cm
	PrimaConnex™/Genesis™ 5.5/6.5 WD	25N. Cm
Megagen	AnyOne® Mini	15N. Cm
	AnyOne® Regular	25N. Cm
	ExFeel	25N. Cm

Manufacturer	Brand	ASC* Recommended Screw Torque Value
MIS	M4/Seven Standard	25N. Cm
	M4/Seven Wide	25N. Cm
Neobiotech	IS Regular	25N. Cm
	IT Regular	25N. Cm
	IT Wide	25N. Cm
Neoss	ProActive Straight 3.25	25N. Cm
	ProActive Straight 3.5, 4.0, 4.5, 5.0, 5.5	25N. Cm
	ProActive Tapered 3.5, 4.0, 4.5, 5.0, 5.5	25N. Cm
Nobel Biocare®	NobelActive™ NP	25N. Cm
	NobelActive™ RP	25N. Cm
	NobelActive™ WP	25N. Cm
	NobelReplace™ NP Conical Connection	25N. Cm
	NobelReplace™ RP Conical Connection	25N. Cm
Straumann®	Bone Level 2.9 SC (Narrow CrossFit™)	25N. Cm
	Bone Level 3.3 NC (Narrow CrossFit™)	25N. Cm
	Bone Level 4.1, 4.8 RC (Regular CrossFit™)	25N. Cm
	Tissue Level synOcta 4.8 RN (Regular Neck)	25N. Cm
	Tissue Level synOcta 4.8 WN (Wide Neck)	25N. Cm
Zimmer® Dental	Zimmer Eztetic™ 3.1	25N. Cm
	Tapered Screw - Vent™ 3.5, 4.5, 5.7	25N. Cm
	Biomet 3i Certain™ 3.4	20N. Cm
	Biomet 3i Certain™ 4.1	20N. Cm
	Biomet 3i Certain™ 5.0	20N. Cm
	Biomet 3i Certain™ 6.0	20N. Cm

Please note: A TRU Torque DRIVER must be used for all TRUAbutment ASC cases. Please contact our team to receive a loaner.

*Angulated Screw Channel (ASC)